


SGTi-flex COVID-19 IgM/IgG

5 tests & finger-prick tools

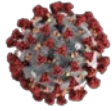

Healthcare professional use only



SGTi-flex COVID-19 IgM/IgG Test is an immunoassay for qualitative detection of IgM or IgG antibodies to COVID-19 in human whole blood. The test is useful as a screening test for COVID-19.

WARNING AND PRECAUTIONS

- For in-vitro diagnostic use only. Healthcare professional use only.
- Clinical diagnosis should be made through a comprehensive review of the specialist based on other test methods and clinical symptoms.

Kit contents

Test kit	Test Cassette	5 EA
	Sample Buffer	1 EA (1ml/tube)
	Instructions for Use	1 EA
Finger-prick tools	Safety Lancet	5 EA
	Alcohol Swab	5 EA
	pettyPip (10ul capillary pipette)	5 EA



- ✓ Rapid testing for SARS-CoV-2 antibodies within 10 minutes
- ✓ Comprehensive Package! No need for additional tool/device
- ✓ Just 10ul of finger-prick blood!
- ✓ Perfect Matrix equivalency

The finger-prick whole blood shows equivalent clinical performance to the venipuncture whole blood and serum.

TOTAL	Venipuncture whole blood			
	Pos	Neg	Total	
Finger-prick whole blood	Pos	30	0	30
	Neg	0	30	0
Total	30	30	60	

Outstanding performance

TOTAL	Real Time RT-PCR			
	Pos	Neg	Total	
SGTi-flex COVID-19 IgM/IgG	Pos	154	6	160
	Neg	9	354	363
Total	163	360	523	

- Sensitivity : 94.48% (95% CI : 89.84%~97.07%)
- Specificity : 98.33% (95% CI : 96.41%~99.23%)

Specification

Information	Detail
Time to result	10 ~ 15 min.
Sample type	Whole blood
Storage	2~30°C (36~86°F)
Catalog No.	COVT005E