AQUAGUARD[®] BOOT

a Covalon Technologies Ltd. Product

INSTRUCTIONS FOR USE

(EN)

Description

AquaGuard[®] Boot[®] is an impermeable moisture barrier that allows a clinician or patient to cover wounds, dressings and intravenous sites and keep them dry when showering. It consists of an occlusive polyethylene barrier with a flexible water-seal band that is adjustable to cover various sites.

EN Shower Protection with Water-Seal Band*

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HR Zaštita navlaka za tuširanje s vodonepropusnim trakom Water-Seal Band*

EN Directions for Application

1. While seated with knee flexed, slide boot over leg and above the site to be protected, leaving some slack.

2. Prepare the Water-Seal[®] Band[®] (band) by pressing and holding the lever down to completely open the buckle. See Figure 1a.

3. Slide the band over boot and above site to be protected.

4. Pull loose end of the band until the buckle is snug against the leg. See Figure 1b.

5. Confirm an effective seal by attempting to insert a finger underneath band.

6. Secure excess band material by wrapping around leg.

7. Fold excess boot material over band to form cuff. See Figure 2.

EN Directions for Removal

1. After shower gently pat cuff and surrounding area dry before removal.

2. Remove the band buckle by pressing and holding lever while drawing buckle away from leg.

3. Release the sutured edge of the band from the cuff.

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EN Notes

• Some dampness may occur inside boot due to perspiration.

• Boot may be trimmed to size prior to application if required.

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FR Indications for Use

AquaGuard[®] Boot[®] is designed to cover and keep wounds, dressings and intravenous sites dry during showering. Common applications include casts, knee or ankle surgery sites, orthopedic sites, diabetic wounds, and other foot or leg sites. This product may be applied by healthcare providers or patients.

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IT Precautions

AquaGuard[®] Boot[®] is intended for showering only and is not recommended for swimming or submersion under water. AquaGuard[®] Boot[®] can be slippery when wet, use with caution.

AquaGuard[®] Boot[®] is a suffocation hazard. Keep out of reach of children. Do not use if punctured or showing signs of wear. AquaGuard[®] Boot[®] is single use product and should be worn over primary dressings, wounds, and other sites during showering.

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